


Prosys®

# Catheter Strap

User guide

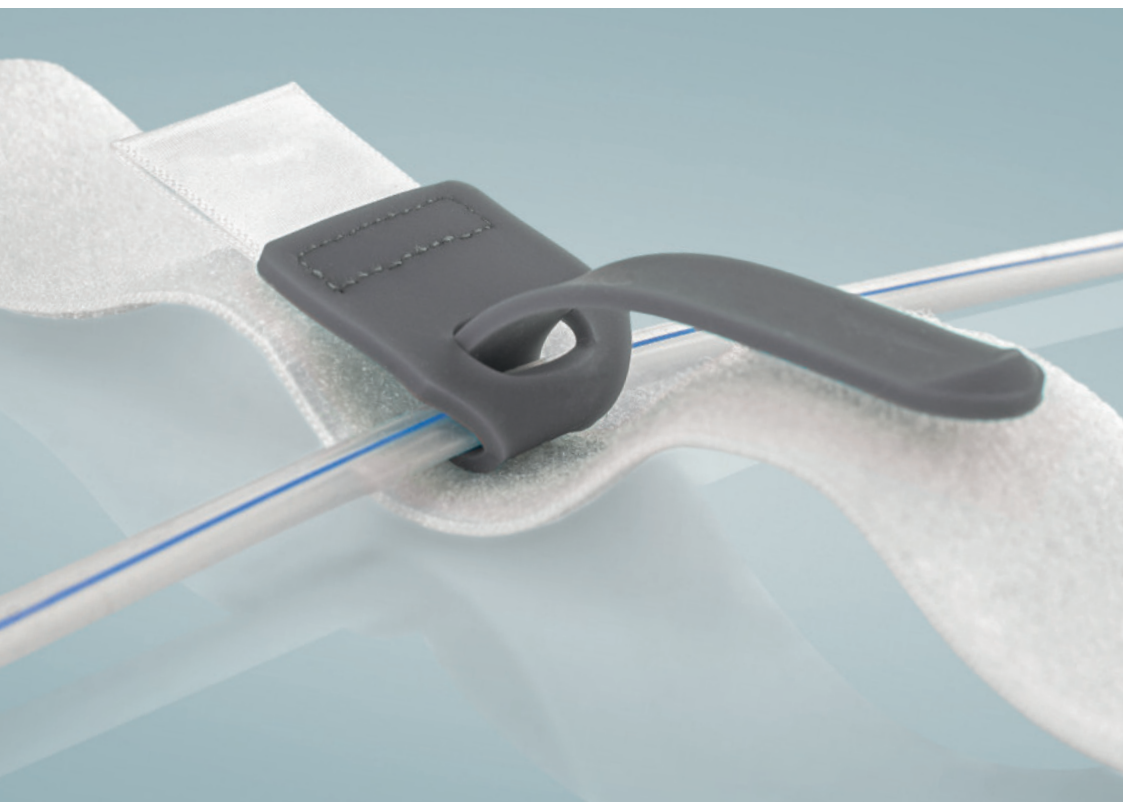

# Designed for you

The Prosys range of continence products have been designed to be comfortable to use and to provide you with confidence in managing your bladder health.

Our simple Prosys Catheter Strap provides all the support you need to maintain an active lifestyle and the freedom to move around safely.

## Prosys Catheter Strap

### Comfort

- ✓ Elasticated design allows for free movement
- ✓ Soft, breathable fabric maximises comfort
- ✓ Cut-to-size to remove any excess material

### Confidence

- ✓ Clasp ensures the catheter stays fixed in position
- ✓ Silicone beading gently secures the strap without irritation
- ✓ Compatible with all indwelling catheters and leg bags

### Any questions?

Call 0800 012 6779 and we will be happy to help




# How to use

The Prosys Catheter Strap is most commonly worn on the leg in the thigh position.

For those with a suprapubic catheter, the strap can be positioned on the lower abdomen.


## Securing the strap to your body

Place the strap around either the leg or lower abdomen. Fasten the strap in the desired position on your leg or abdomen. Ensure the strap is snug and the clasp is easily accessible.

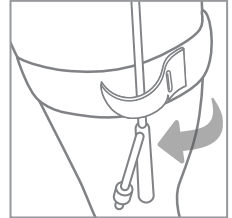

## Securing the catheter tube

Position the clasp over the tubing. Wrap the clasp around and under the tubing. Push the tapered end of the clasp through the eyelet provided.

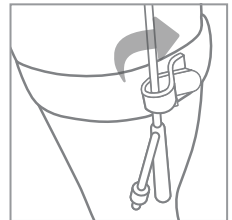

## Securing the clasp

Gently pull the clasp to tighten around the tube. Pull the clasp up and over, pressing the tapered end into place and securing it on the main strap.



Prosys Catheter Straps:



# Use the Catheter strap to secure either the catheter or the leg bag tube



National Institute for Health and Care Excellence (2012) Infection: Prevention and Control of Healthcare-Associated Infections in Primary and Community care.\*

## Caring for your strap

To support performance and good hygiene, we recommend you wash the strap. To do so, secure the clasp in place onto the strap and wash according to the instructions on the label.



\*Clinical Guideline 139. NICE, London

# Ordering information

## Prosys Catheter Strap

Reference	Size	Length	Quantity	PIP code
PCS40	Short	40cm	5	4085536
PCS50	Adult	50cm	5	4085544
PCS85	Abdominal	85cm	5	4085551




# Let us come to **you**

Simply order your NHS prescriptions for any urine bags, catheters or sheaths and we will deliver direct to your door, at no extra cost to you or the NHS.



Dedicated customer care line

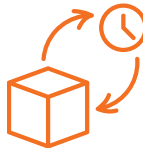

Every order tracked to your door



Every order is checked and re-checked to ensure order accuracy

Call our customer care team:

**0800 012 6779**

Opening hours:  
8am-6pm Mon-Fri. 9am-12pm Sat.

Anytime online:

**www.clinidirect.co.uk**

Clinisupplies product support: 0800 066 3754

Email: [productsupport@clinisupplies.co.uk](mailto:productsupport@clinisupplies.co.uk)

People love us

★ Trustpilot

★★★★★ Excellent\*

\*October 2022