



Libra[®] Leg Bag Holder

Instructions for use

Indication

Medical device for suspension of a leg bag.

Features

The Libra Leg Bag Holder provides a pocket to hold a urinary leg bag in place. It has an opening in the bottom for the drainage tap of the leg bag to facilitate drainage without the need to remove the leg bag holder. It is suitable for use on either the calf or thigh and is designed to be used with drainage bags that have a volume of up to 750ml.

Made from: 53% Polyester 30% Cotton 17% Elastane.

Instructions for use



- 1) Pull the Libra Leg Bag Holder over the foot – as you would do a sock – and pull into place over your calf or thigh
- 2) Ensure that the coloured strip is at the top and facing outwards
- 3) Slide your leg bag into the top of the Libra Leg Bag Holder through the opening in the material
- 4) Push the drainage tap through the hole in the bottom of the pocket
- 5) Adjust the holder as necessary for your comfort



A UK manufacturer committed to Urology.

Washing Instructions

For optimal durability, the Libra Leg Bag Holder should be washed at 60°C in a garment bag. If tumble drying, ensure to use a gentle setting. Do not iron. Use of chlorine-based bleach or chlorine-based disinfectant will reduce durability.

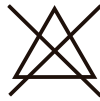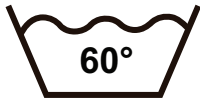

Precautions

If you have any questions about the correct use or fitting, or if any unexpected discomfort or skin irritation occurs while using the Libra Leg Bag Holder, please consult your healthcare professional.

Order Codes

Re-Order Code	Colour Band	Size	Quantity
10660S	Yellow	Small (30-45cm)	4
10655M	Blue	Medium (45-65cm)	4
10654L	Brown	Large (55-80cm)	4
10665X	Green	Extra Large (65-90cm)	4



STANDARD
100
6234-7001
DTI

 Great Bear Healthcare Ltd
1 Lambourne Crescent,
Cardiff Business Park,
Cardiff, CF14 5GF
T +44 (0) 29 2074 7770

mail@gbhl.co.uk
www.greatbearhealthcare.co.uk

EC REP

MediReps Ltd
First Floor, Penrose One,
Penrose Dock, Cork City,
T23 KW81,
Ireland

LIBRALBHIFUV3/2023
P1894.03-03-2023