



Putting Patients First

### Products available through Nightingale Home Delivery Service

1. Call **Freephone 0800 304 7434**
2. Please ensure you have your GP details to hand
3. Tell us which products are required and how many

**That's all – we will do everything else!**

Please make sure you have at least two week's supply in hand before re-ordering.

#### Washing Instructions

- 1) The Paediatric Strap can be handwashed at 30°. It is not suitable for machine wash
- 2) Do not Iron or Tumbledry



Latex Free



# Paediatric Strap

## Instructions For Use



**Great Bear Healthcare Ltd**  
 1 Lambourne Crescent,  
 Cardiff Business Park,  
 Cardiff, CF14 5GF  
 T +44 (0) 29 2074 7770

mail@gbhl.co.uk  
 www.greatbearhealthcare.co.uk



**MediReps Ltd**

First Floor, Penrose One,  
 Penrose Dock, Cork City,  
 T23 KW81,  
 Ireland

MBBEARSTRAP1FUV3/2023




A UK manufacturer committed to Urology.

## Your Paediatric Leg Bag Strap

The Paediatric Strap is designed specifically to secure the Paediatric Leg Bag to the leg.

The centrally positioned design and strong Velcro closure keeps the Paediatric Bag secure without hindering movement.

**Important: Do not let the Leg Bag hang unsupported from the Catheter as this will cause urethral discomfort and trauma.**

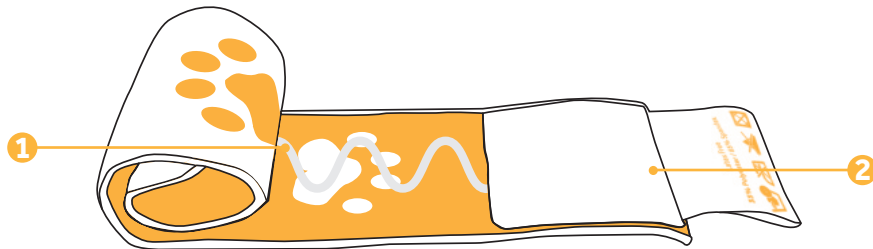

### Key

1 Silicone Lines

2 Velcro Tab

### Before Use



- 1) Measure the circumference of the leg (where the Paediatric Bag is going to be positioned) and ensure you have the right size strap from the table below
- 2) If the measurement falls between the two sizes, select the longer strap and cut to size

## Order Codes

Order Code	Strap Length	Qty
10633P	30cm	5
10657P	50cm	5

## Instructions For Use

- 1) Feed the strap through the eyelets at the side of the Paediatric Leg Bag (Diagram 1)
- 2) Ensure the silicone lines(1) are facing the skin
- 3) Wrap the strap around the thigh or calf in the desired position (Diagram 2)
- 4) Secure using the velcro tabs(2). Ensure the strap is not overtightened around the leg as this can affect circulation

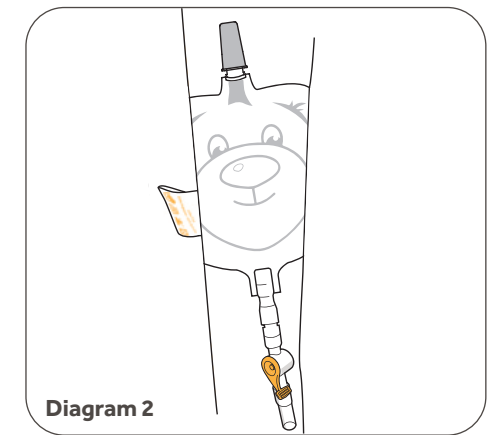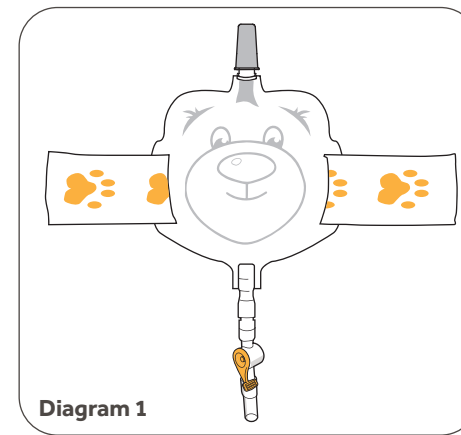

### Cautions

- Use of any medical device carries associated risks. Device failures are rare, however in the event of a device failure of this product, you should contact Great Bear Healthcare using the details provided within this leaflet
- If a failure has resulted in a serious incident, such as significant deterioration of health or death, you should contact Great Bear Healthcare and your local Competent Authority as soon as possible. Details of your local Competent Authority can be found on your local government website