

Designed for you

The Prosys® range of continence products have been designed to be comfortable to use and to provide you with confidence in managing your bladder health.

Our comprehensive range of Prosys® All-Silicone Foley Catheters are available in multiple sizes and two different tips and will sit comfortably in situ for up to 90 days.

Prosys® All-Silicone Foley Catheter

For Comfort


- ✓ Made with 100% silicone
- ✓ Flexible tubing
- ✓ Compatible with all urine drainage bags

For Confidence

- ✓ Large rounded eyelets for optimal drainage
- ✓ Open Ended tip for reduced risk of blockages
- ✓ Nelaton tip to facilitate catheter insertion

Any questions?

Call 0800 012 6779 and we will be happy to help

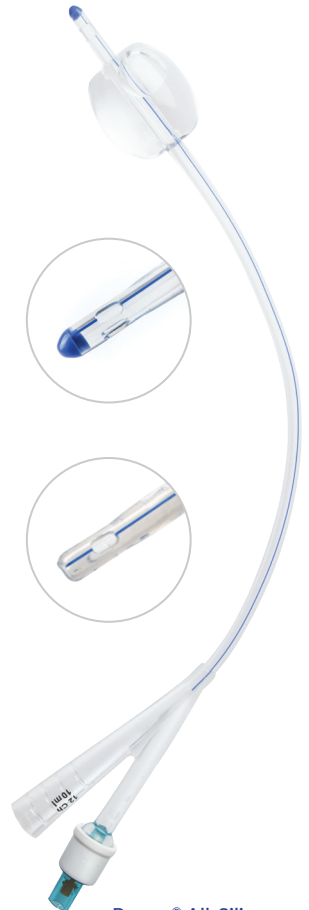

Prosys® All-Silicone
Foley Catheter:



Connecting to Prosys[®] Leg Bag

- ① To reduce the risk of infection, it is important you wash your hands prior to connecting the Prosys[®] Leg Bag to the Prosys[®] All-Silicone Foley Catheter.
- ② Ensure the lever on the outlet tap is in the closed position.
- ③ Remove the blue protective cap from the universal connector.
- ④ Insert the connector into the end of the indwelling catheter using a twist motion to ensure a secure connection.



Make sure you do not touch the catheter or leg bag connectors as that will increase chances of cross-contamination and infection.

How to secure

- ① The Prosys[®] Leg Bag should be positioned on the thigh or calf, depending on the tube length, and secured appropriately with the leg bag straps provided.
- ② Feed the Prosys[®] Leg Bag Straps through the eyelets on the bag and ensure the silicone beaded lines are positioned against the leg for optimal comfort and security.



Caring for my indwelling catheter

- ① Ensure that you wash your hands before and after handling your catheter, urine drainage bag or catheter valve.
- ② Wash the area where your catheter enters the body with mild unperfumed soap and warm water at least once a day.
- ③ Do not use oil-based creams around the catheter.
- ④ Avoid unnecessary disconnection of the leg bag and do not touch the connector when replacing. This will help to prevent infection.
- ⑤ Ensure the drainage bag is kept below the level of your bladder to promote urine drainage.
- ⑥ To maintain urine flow, ensure your catheter tubing is not kinked.



The National Institute for Health and Care Excellence, recommend you secure your catheter with a catheter strap to prevent the risk of urethral trauma.



Ordering information

Prosys® All-Silicone Foley Catheter

Long-term, 2-way with 10ml balloon and pre-filled syringe (sterile water)

Box of 1

| Reference | Description | Size | Length | PIP code |
|-----------|-----------------|--------|--------|----------|
| PCF12M10 | Standard length | ● 12Ch | 40cm | 386-0616 |
| PCF14M10 | Standard length | ● 14Ch | 40cm | 386-0582 |
| PCF16M10 | Standard length | ● 16Ch | 40cm | 386-0574 |
| PCF18M10 | Standard length | ● 18Ch | 40cm | 386-0541 |
| PCF12F10 | Female length | ● 12Ch | 26cm | 386-0624 |
| PCF14F10 | Female length | ● 14Ch | 26cm | 386-0558 |
| PCF16F10 | Female length | ● 16Ch | 26cm | 386-0608 |
| PCF18F10 | Female length | ● 18Ch | 26cm | 386-0590 |

Prosys® Open Ended Foley Catheter

Long-term, 2-way with 10ml balloon and pre-filled syringe (sterile water)

Box of 1

| Reference | Description | Size | Length | PIP code |
|-----------|-----------------|--------|--------|----------|
| PCF12M10E | Standard length | ● 12Ch | 40cm | 419-0674 |
| PCF14M10E | Standard length | ● 14Ch | 40cm | 419-0682 |
| PCF16M10E | Standard length | ● 16Ch | 40cm | 419-0690 |
| PCF18M10E | Standard length | ● 18Ch | 40cm | 419-1086 |

Prosys® Hydro-Caine® Lubricating Gel

2.00% Lidocaine Hydrochloride, 0.05% Chlorhexidine Gluconate Solution

Box of 10

| Reference | Description | Size | PIP code |
|-----------|-------------|------|----------|
| HYD006 | Small | 6ml | 366-5528 |
| HYD011 | Large | 11ml | 366-5510 |

Prosys® Catheter Strap


| Reference | Description | Length | Quantity | PIP code |
|-----------|-------------|--------|----------|----------|
| PCS40 | Short | 40cm | 5 | 408-5536 |
| PCS50 | Adult | 50cm | 5 | 408-5544 |
| PCS85 | Abdominal | 85cm | 5 | 408-5551 |

Let us come to **you**

Simply order your NHS prescriptions for any urine bags, catheters or sheaths and we will deliver direct to your door, at no extra cost to you or the NHS.



Dedicated customer care line

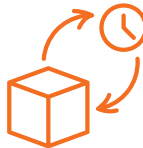

Every order tracked to your door



Every order is checked and re-checked to ensure order accuracy

Call our customer care team:

0800 012 6779

Opening hours:
8am-6pm Mon-Fri. 9am-12pm Sat.

Anytime online:

www.clinidirect.co.uk

Clinisupplies product support: 0800 066 3754

Email: productsupport@clinisupplies.co.uk

People love us

★ Trustpilot



Rated Excellent*

*Feb 2023

Prosys®

All-Silicone Foley

User guide

